

Basic Catheterisation Trainers Value Set Product Guide



See the Setup Guide for additional information



Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK.
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
sales@limbsandthings.com www.limbsandthings.com

Issue 1, March 2011
© 2011 Limbs & Things

Basic Catheterisation Trainers - Value Set

Part No: **60166**

The Basic Catheterisation Trainers allow the trainee to learn urethral catheterisation. They have accurate male and female anatomy, and both sets of genitalia are realistically soft and compliant. The feel of the catheter passing along the urethra into the bladder corresponds closely to real life.

Skills

- Correct handling of male and female anatomy
- Aseptic catheterisation technique
- Catheter placement: accepts catheters from 14-16 French (male) and 12-16 French (female)
- Fluid management
- Withdrawal of catheter
- Intermittent self-catheterisation (ISC) when using the optional **Self Catheterisation Stand** Part No: **60167**

Features

- Latex free
- Supple urethra and resistant sphincter providing realistic response
- Non drip valve
- Male anatomy: realistic meatus, flaccid penis and replaceable foreskin
- Female anatomy: partable labia with realistic urethral meatus and vaginal opening
- 1 litre fluid bag on stand provides pressurised fluid flow
- Syringe supplied with water-based lubricant to simulate proprietary local anaesthetic gel

Package supplied


- 1 **Male Catheterisation Unit**
Part No: **60152**
- 1 **Catheterisation Foreskin (Pack of 3)** Part No: **60154**
- 1 **Standard Bung** Part No: **60161**
- 1 **Antifungal Agent (250ml)**
Part No: **50185**
- 2 **Aseptic Catheterisation Sleeves**
Part No: **60160**
- 1 **Mock LA Gel with Syringe**
Part No: **50183**
- 1 **Male Catheterisation Sphincter Kit** Part No: **60159**
- 1 pelvic shell
- 1 drip tray, bag stand, 1 litre fluid bag
- 1 plastic kidney dish
- 1 14 French foley catheter (male)
- 1 2 litre jug
- 1 **Female Catheterisation Unit**
Part No: **60156**
- 1 **Standard Bung** Part No: **60161**
- 1 **Antifungal Agent (250ml)**
Part No: **50185**
- 2 **Aseptic Catheterisation Sleeves**
Part No: **60160**
- 1 **Mock LA Gel with Syringe**
Part No: **50183**
- 1 pelvic shell
- 1 drip tray, bag stand, 1 litre fluid bag
- 1 plastic kidney dish
- 1 12 French foley catheter (female)
- 1 2 litre jug

Components



Female Catheterisation Unit

Part No: **60156**

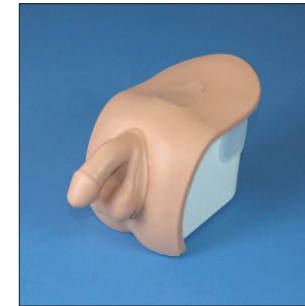

Male Catheterisation Unit

Part No: **60152**



Catheterisation Foreskin (Pack of 3)

Part No: **60154**



Standard Bung

Part No: **60161**



Antifungal Agent (250ml)

Part No: **50185**



Aseptic Catheterisation Sleeves

Part No: **60160**



Mock LA Gel with Syringe

Part No: **50183**



Male Catheterisation Sphincter Kit

Part No: **60159**


Note

Do not leave catheters inserted in the Trainer or attempt to remove a catheter which has an inflated balloon.

Catheterisation is a wet process. Do not set up the product near electrical appliances or items damaged by water.

Attaching the Foreskin


1

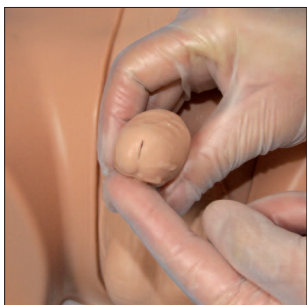

Apply the supplied water-based lubricant to the glans of the penis.

Ensure the entire glans is coated.



2

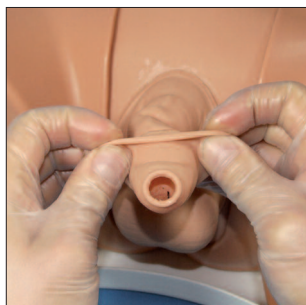

Ensure the correct orientation of the Foreskin.

Carefully roll back half the proximal end of the Foreskin, and feed it over the glans and onto the shaft of the penis.

3

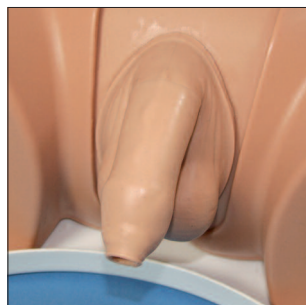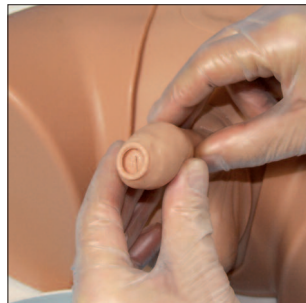



Push the Foreskin back so that the opening (distal end) is close to the tip of the glans.

Ensure the Foreskin fits smoothly over the penis.

The penis is now ready for catheterisation.

Using the optional Self Catheterisation Stand


1

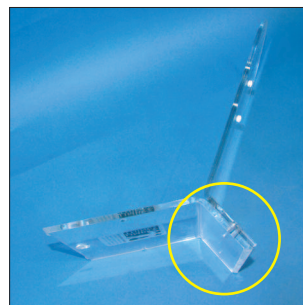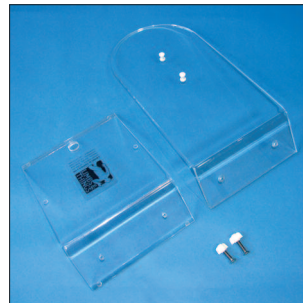

The **Self Catheterisation Stand** Part No: **60167** comprises a Base Plate and a Back Plate (with two white pegs) which are fastened together with two screws.

Stand the Plates on their edges (on the work surface) and align the two short panels with the corresponding holes together.


2

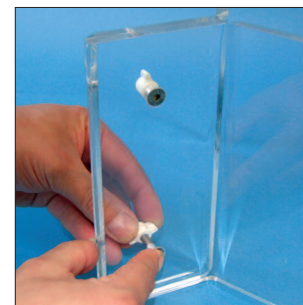

Ensure the screw holes line up, and insert the screws.

The head of each screw should be flush with the surface of the Back Plate. Hold each screw in place with a finger and tighten the white wingnut.


3

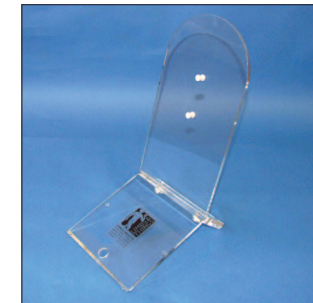

Place the Stand in the upright position.

Align the keyholes on the bottom of the pelvic base with the two white pins on the Back Plate.

4

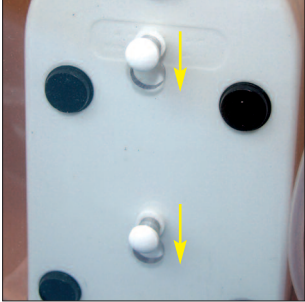

Drop the pelvic base down onto the pins so that it locks into position on the Back Plate.

The Catheterisation Trainer is now ready for demonstrating and practising self catheterisation.

Visit www.limbsandthings.com

- For the latest product information
- To see the product demonstration video
- To download a printable PDF of this Product Guide and the Setup Guide
- For the latest news and events
- For information on the comprehensive range of products that we stock

Limbs & Things UK
Skills training products for healthcare professionals

Products Resources News & Events About Us Support
Physical Examination Skills General Procedural Skills Specialist Skills Clinical Skills: Product Matrix

Home > General Procedural Skills > Urinary Catheterisation > Basic Catheterisation Trainer Value Set

Basic Catheterisation Trainer Value Set
PART 60166

Description Features Package Contents User Manuals

In response to customer demand, we have introduced a cost effective Value Set containing a basic version of each of the Male and Female Catheterisation Trainers. These basic versions do not contain the **suprapubic bungs** or the self-catheterisation stands (although these can be ordered separately if wished at a later stage).

The Catheterisation Trainers allows the trainee to learn urinary catheterisation techniques. The feel of the catheter passing along the urethra into the bladder corresponds closely to real life.

Skills

- Correct handling of male and female anatomy
- Aseptic catheterisation technique
- Catheter placement: accepts catheters from 14-16 French (male) and 12-16 French (female)
- Fluid management
- Withdrawal of catheter

VIDEO
Advanced Catheterisation Trainer Video
[WATCH](#)