

FGM Deinfibulation Trainer User Guide



Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK.
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
sales@limbsandthings.com www.limbsandthings.com

Issue 1, June 2010
© 2010 Limbs & Things

FGM Deinfibulation Trainer

Part No: **60470**

This unique model has been designed to help recognise female genital mutilation (FGM) and perform the surgical reversal procedure of deinfibulation. This model represents Type III FGM.

The idea of adapting the Keele & Staffs Episiotomy Repair Trainer for deinfibulation training was conceived by Mr Abdalla E A Yagoub (Special Registrar in Obstetrics & Gynaecology, West Midlands Deanery) and developed by him in collaboration with Mr Khaled M K Ismail and Miss Fidelma O'Mahony from the University Hospital of North Staffordshire.

Skills

- Recognise correct anatomy and classification type of FGM
- Identification of important anatomical landmarks
- Guarding vital anatomy whilst performing a surgical incision
- Suturing skills including knot tying

Features

- Skin layers have realistic retention of sutures
- Finger can be inserted to 'guard' the vagina and clitoris whilst incisions are made
- **FGM Deinfibulation Upgrade Kit** Part No: **60473** fits onto the frame supplied with both the Keele & Staffs Episiotomy Repair Trainer and the Sultan Anal Sphincter Trainer

Package supplied

- 1 **FGM Deinfibulation Perineum** Part No: **60471**
- 1 **FGM Deinfibulation Insert Block** Part No: **60474**
- 1 **FGM Deinfibulation Perineal Skin (Pack of 3)** Part No: **60472**
- 1 Frame
- 2 bench top clamps

Components




FGM Deinfibulation Perineum

Part No: **60471**



FGM Deinfibulation Insert Block

Part No: **60474**

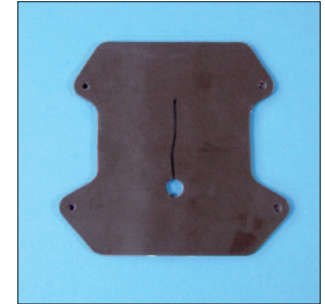

FGM Deinfibulation Perineal Skin (Pack of 3)

Part No: **60472**



Frame



Bench top clamps

Setting up the FGM Deinfibulation Trainer



1



Place the pink layer of the Perineal Skin centrally onto the Perineum.



Ensure that the Skin is the correct way round: with the white plastic strips on the underside.

2

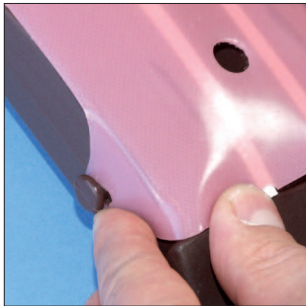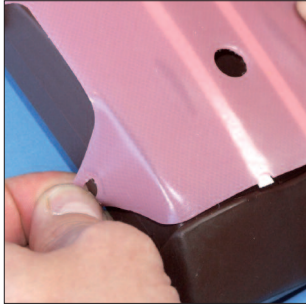


Hold the Skin in place and attach the lower left corner.

3

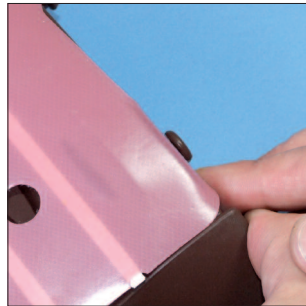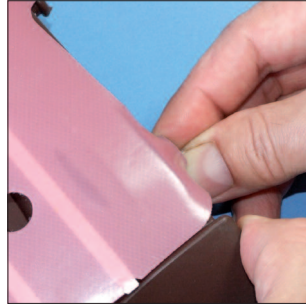

Then attach the lower right.
(Using this left/right method ensures that the Skin stays centrally placed on the Perineum).



4

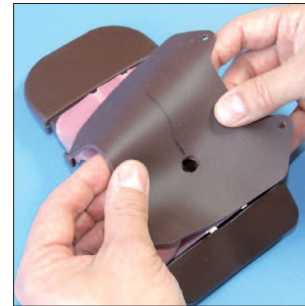

Do the same for the top fixings.

Now place the dark layer onto the pink and make sure that the holes line up.

5

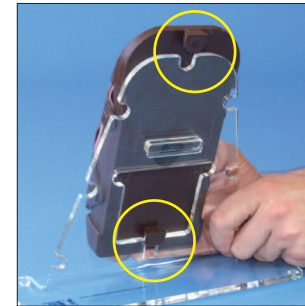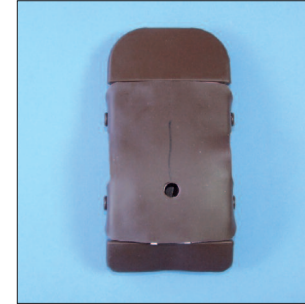

Attach the Skin to the Perineum using the left/right method.

Attach the Perineum to the Frame using the two slots.

6



Attach the Trainer to the edge of the work surface using the supplied clamps.

The Trainer is ready to use.