

PROMPT Birthing Simulator

Upgrade to fit perineum and abdomen

1. Remove the perineal muscles
2. Remove the perineum

Refer to the product user guide if you are unsure of these steps

3. Identify the 2 small plastic screws located on either side of the perineal muscles



4. Using a Stanley knife cut the plastic screw off at its base




5. Once cut off, ensure the surface is flush and clean




6. Take the supplied superglue and one blanking plug



7. Apply superglue to the back of the blanking plug

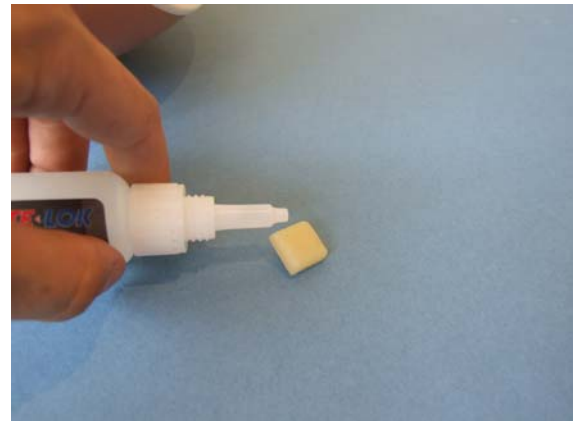

8. Fit the blanking plug into position

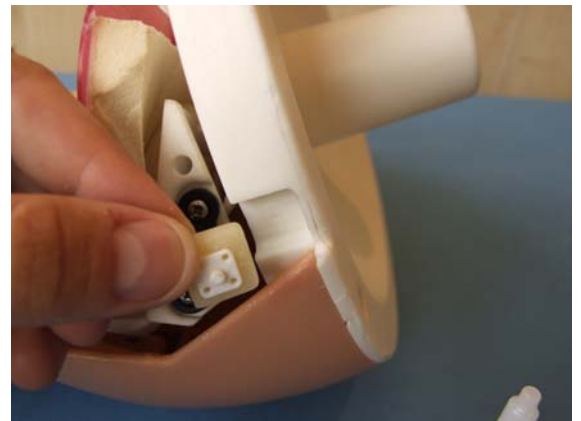

9. Push and hold the blanking plug in place until secure

10. Repeat the process for the other side

11. Re-fit the perineum and perineal muscles.

PROMPT Birthing Simulator is ready to use

