

Male/Female Catheterisation Sphincter Kit User Guide



For use with:

- Advanced Catheterisation Trainer** Part No: **60150**
- Advanced Male Catheterisation Trainer** Part No: **60151**
- Advanced Female Catheterisation Trainer** Part No: **60155**
- Basic Catheterisation Trainers - Value Set** Part No: **60166**



Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK.
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
sales@limbsandthings.com www.limbsandthings.com

Issue 2, July 2011
© 2011 Limbs & Things

Male Catheterisation Sphincter Kit

Part No: **60159**



Package supplied

- Male sphincter
- O ring
- Lubricant

Female Catheterisation Sphincter Kit

Part No: **60162**




Package supplied

- Female sphincter
- O ring
- Lubricant

Note


From 1st August 2011 the Sphincter Kits are no longer supplied with an allen key for the removal and refitting of the pubic bone. The procedure has been simplified and the pubic bone now stays in place. This revised procedure is compatible with all Basic and Advanced Catheterisation Trainers.

To be used with the following models:

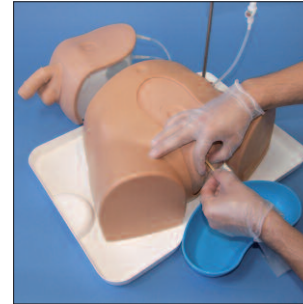

Advanced Catheterisation Trainer

Part No: **60150**

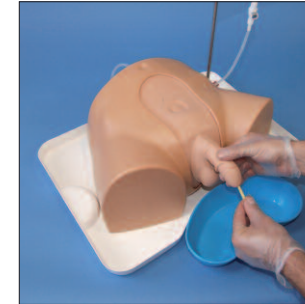

Advanced Male Catheterisation Trainer

Part No: **60151**



Advanced Female Catheterisation Trainer

Part No: **60155**



Basic Catheterisation Trainers - Value Set

Part No: **60166**

Removing the sphincter

1



Place the Catheterisation Unit on the work surface and remove the Bung.

Lift up the Skin at the back and release it from the base.


2



Peel the Skin away from the top of the base and allow it to hang down over the front of the Unit.

Hold the base down. Grip the Skin securely around the connector and gently pull it away from the bladder.


3

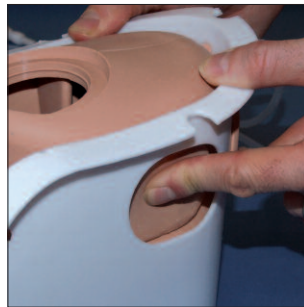

NOTE: When separating the Skin from the bladder, hold the Skin by the thick reinforced area around the connector.

Push in the lugs on either side ...



4

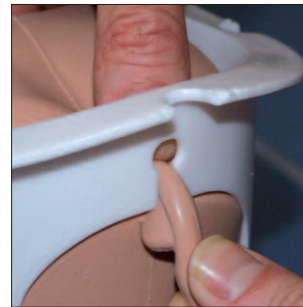

... and on the back of the bladder to separate it from the base.

Push the fluid tube out of the slot at the back of the base.

5

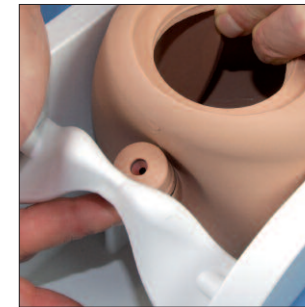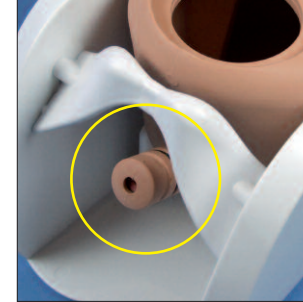

Push the valve housing...

... up and behind the pubic bone.

6





Push the bladder up and out of the base.

Make sure the bladder is completely empty, then place it upside down on the work surface.

Replacing the sphincter



7

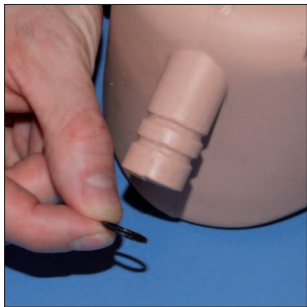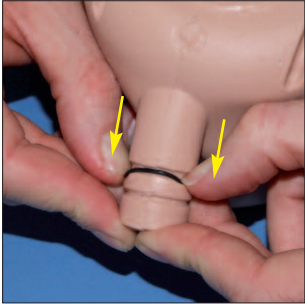

Using both thumbs, push the O ring out of the groove ...

... and off the end of the urethral connector.

8

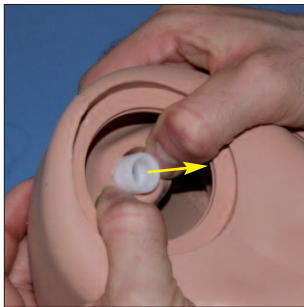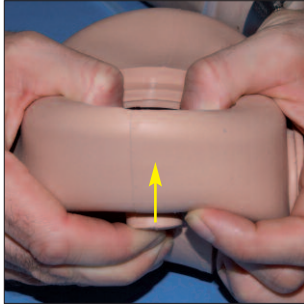

Turn the bladder back over and push the urethral connector up into the bladder.


The sphincter should start coming out of the connector.

9



Continue to push ...
... until the sphincter is free.



1

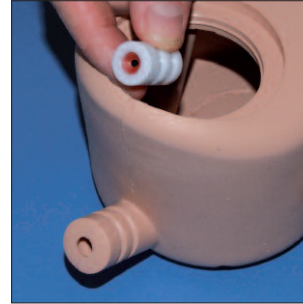

When inserting the new sphincter ensure that the end with the red valve faces outwards.

The male and female sphincters are easy to differentiate: the female sphincter (top picture) is shorter in length than the male (bottom picture).


2

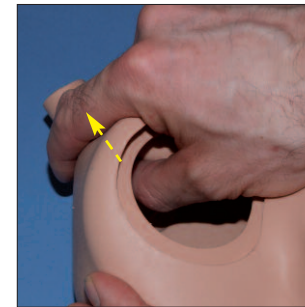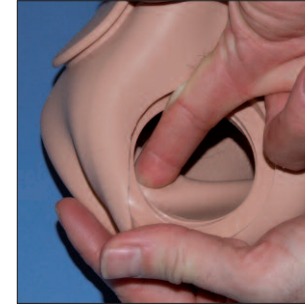

Lubricate the inside of the urethral connector.

Insert the sphincter into the connector and push it in with your thumb.



3

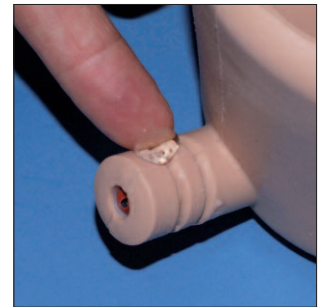

Push the sphincter in until the red valve is visible at the end.

Lubricate the outer part of the connector.

4

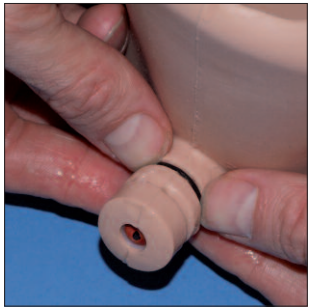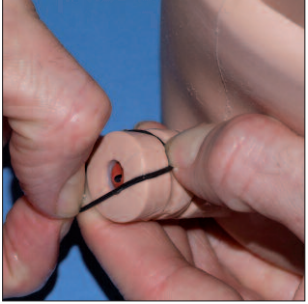

Slide the O ring over the end ...

... and onto the innermost groove.



5



Feed the fluid tubes through the hole at the back of the base...

...and then gently push the bladder back into the base.

6

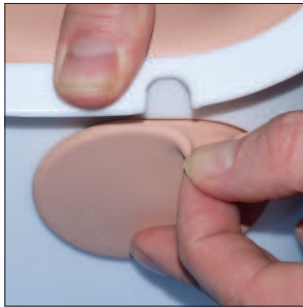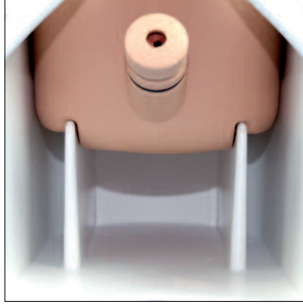

Ensure that the bottom of the bladder fits into the fins on the bottom of the base

Gently push the lugs into the holes in the side of the base to secure the bladder in position.


7



Lubricate the inside of the urethral connector on the Skin.

Use 2 or 3 fingers to support the back of the sphincter.

8

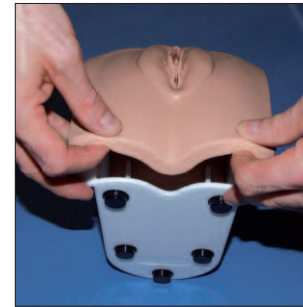

Line the bladder up with the Skin and push them together. A 'pop' will be felt when the parts are correctly fitted.

Attach the bottom of the Catheterisation Skin to the base.

9




Secure the Skin around the lip at the front of the base.

Drape the Skin over the top of the base and secure it with the lug at the back.

Notes


10

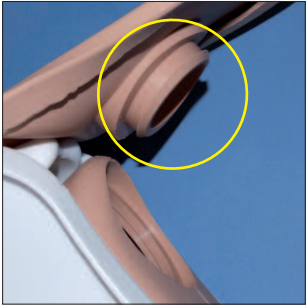

Secure the Skin with the lugs at the sides.

There is a lip around the edge of the bung hole on the Skin ...

11

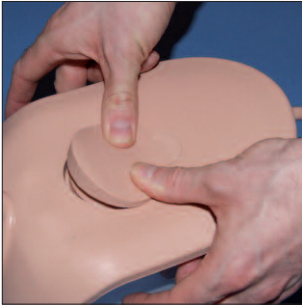

... ensure that the lip fits properly into the bung hole of the bladder.

Lubricate the bung and push it back into position.

The Catheterisation Unit is now ready to use.