

Clinical Female Pelvic Trainer (CFPT) - Standard



Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK.
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
sales@limbsandthings.com www.limbsandthings.com

Issue 2, March 2007
© 2007 Limbs & Things

For more skills training products visit

www.limbsandthings.com

Clinical Female Pelvic Trainer (CFPT) Mk 2

Part No: **60800**

Presents accurate anatomical and tactile representation of the female pelvis for 'hands on' examination and diagnosis of pathologies and abnormalities. It can be used for many levels of training from undergraduate onwards as well as for family health. The uncluttered design of this model enables trainers to concentrate on essential anatomy and procedure.

Skills

- Recognition of anatomy and appropriate landmarks
- Digital vaginal examination procedure
- Bi-manual examination
- Cervical smear procedure (including use of speculum)
- Digital rectal examination procedure

Features

- Interchangeable uterine assemblies with different complications (*order separately*)
- Interchangeable uteri have uterine cavity and cervix
- Realistic abdominal wall
- Partial thighs aid anatomical orientation
- Abdominal wall unclips and lifts off
- Removable abdominal fat pad to represent different thicknesses of abdominal tissue
- Soft perineum and labia
- Realistic anus and lower bowel
- Freedom to present different training scenarios at appropriate levels
- The surface of each foam component is washable using soap and water

Package supplied



- 1 **Female Abdominal Wall Mk 2** Part No: **60810**
- 1 **Clinical Uterus with Normal Anatomy**
Part No: **50117**
- 1 **Silicone Oil (50ml)** Part No: **50180**
- 1 **Helix Connectors (Pack of 10)** Part No: **90100**
- 1 base
- 1 fat pad

Care of products made from Latex Foam Rubber

- Latex foam is a natural product; store away from strong light and in an airtight bag to reduce exposure to ozone. Ozone is generated by some electrical equipment including telephones and computers.
- As when examining a real patient, best practise should be observed with surgical gloves being worn when using the product. Should you need to handle the Trainer with your bare hands, wash them before touching the foam. (If the foam is handled after contact with certain metals such as copper, it becomes yellow and discoloured).
- The foam may be washed effectively using any mild soap and rinsed afterwards. 'Vanish'™ soap is very effective.
- Do not ingest the product and practice normal hygiene procedures after handling the products.

Removing the Female Pelvic Skin


1

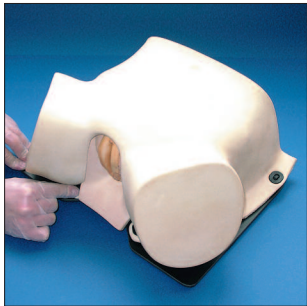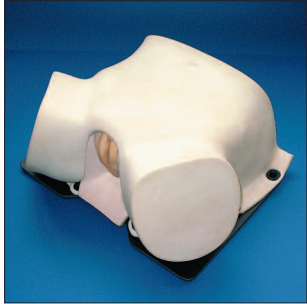

Place the CFPT on the bench top.

Undo the black turnclips



2

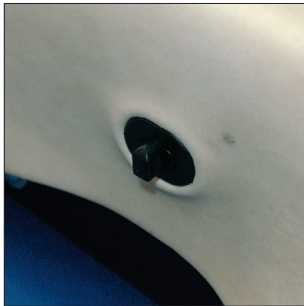

There are 6 turnclips around the edge of the Pelvic Skin.

There is another turnclip at the back of the Skin.

3

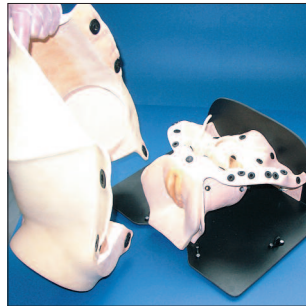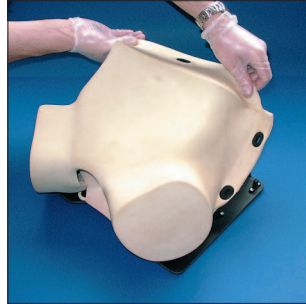



Lift up the Skin from the back of the model.

Lift the Skin off away from the model to reveal the pelvic shell on the base.

1

Changing the Clinical Uterus



1

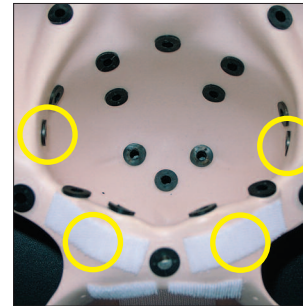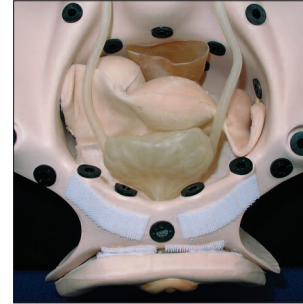

With the Pelvic Skin removed the Clinical Uterus assembly can be clearly seen.

There are 4 pegs (2 on either side of the pelvic shell) which need to be removed first.


2

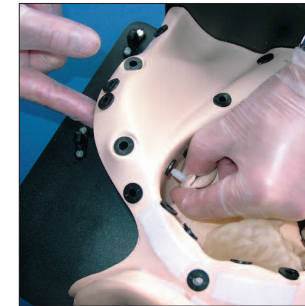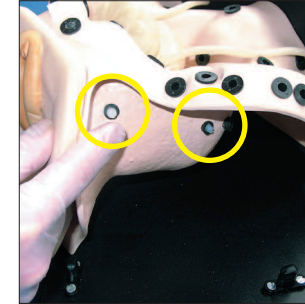

Push the pegs out from one side...

...and then the other.

3

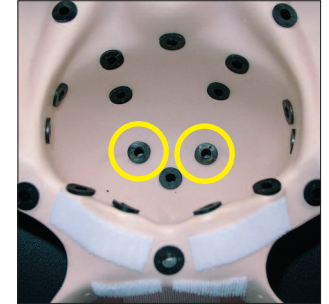


Lift up the uterus to reveal 2 helix connectors at the bottom of the pelvic bowl.

Unscrew each of the connectors.

2



4

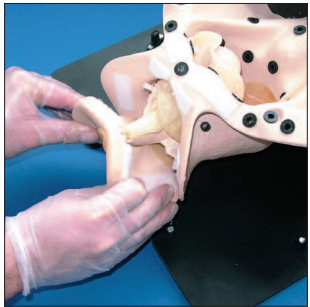

Peel the perineum away from the top of the perineal aperture.

Continue around the edges of the aperture until the perineum is completely free.

5

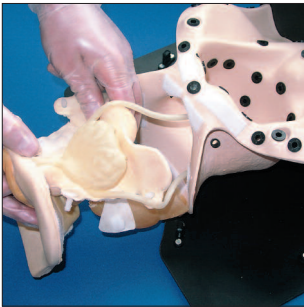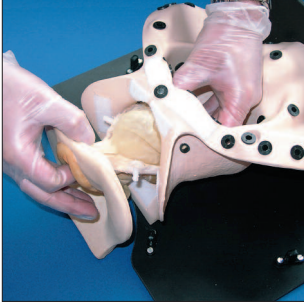

Gently hold the various parts of the uterus and bladder together, and push them through the aperture.

Pull the entire assembly through the aperture and away from the pelvic shell.

To fit a new Clinical Uterus reverse the procedure described in this section.

Interchangeable Uteri available for CFPT:



Clinical Uterus with Ovarian Cyst

Part No: **60360**



Clinical Uterus with Fibroid

Part No: **60361**



Clinical Retroverted Uterus

Part No: **60362**