


Diagnostic Face with Lesions

- This face **20001** is an invaluable teaching aid for the recognition of a wide variety of skin lesions; it provides a basis for discussion of suitable procedures for their treatment and the planning of skin flaps and excision.


The lesions:

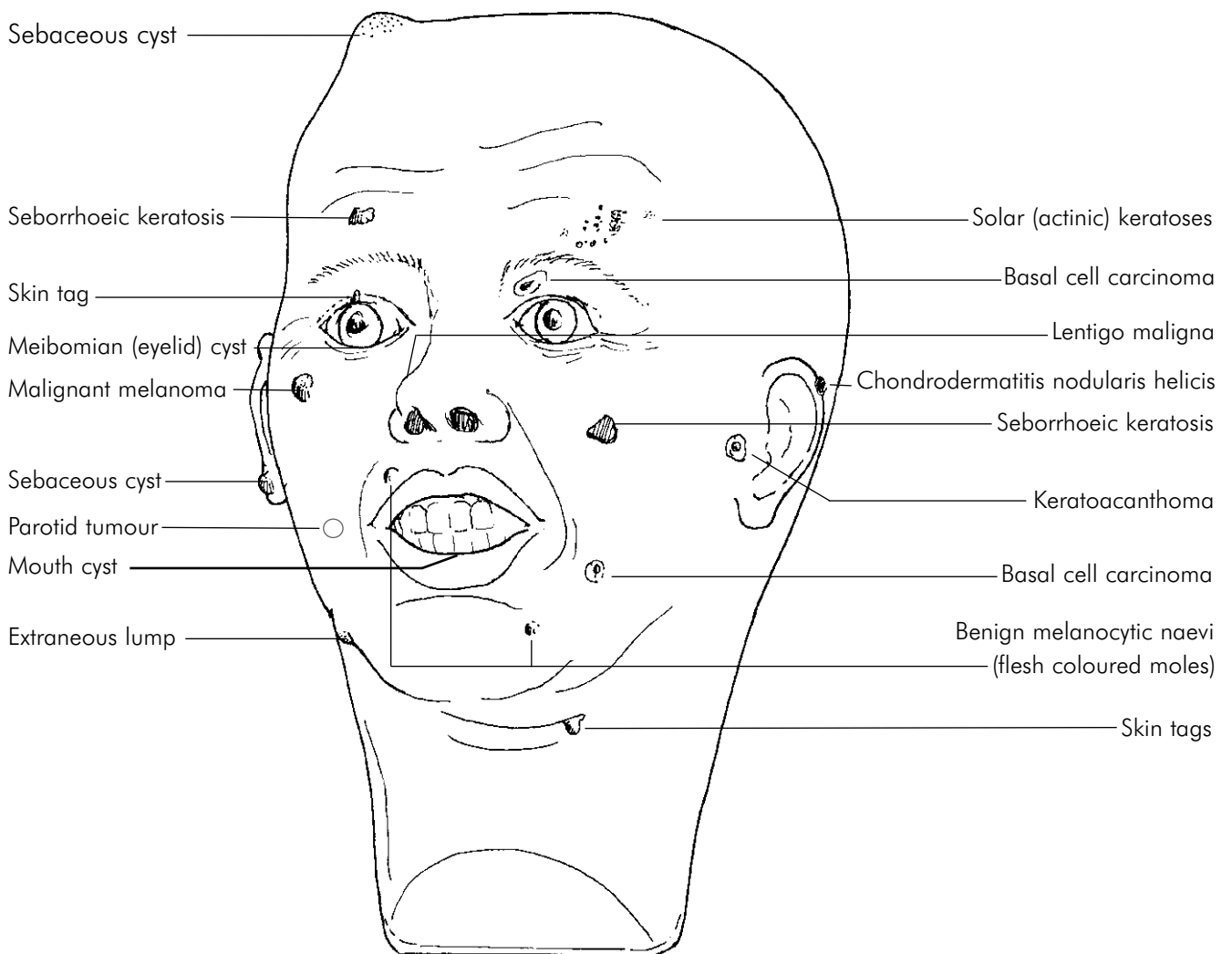

During the training programme the trainees can be encouraged to draw on the model using the special pen provided, the mark of which will fade within 24 hours. Muscle direction, nerve paths, the position of the flaps, skin creases and so on can be drawn on the model without any worry of spoiling it.

Further training:

Following the diagnostic training session on the face, the Curettage, Snip & Shave Excision Pad (Large **00103**, Small **00104**) can be used in the Skin Pad Jig Mk 3 **00550** for practising superficial dermatological lesion removal techniques.

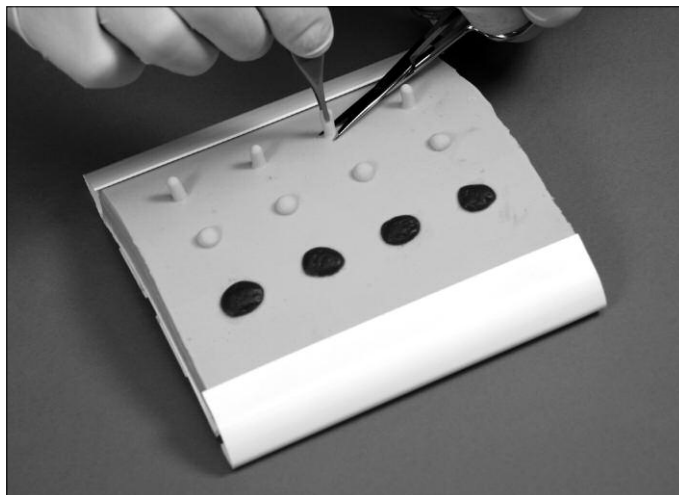

Curettage

The seborrhoeic keratoses are designed to be scraped away using a curette and not by incision.

Snip Excision

The skin tags are designed to be lifted using forceps and removed using surgical scissors.

Shave Excision

The compound naevi are designed to be lifted with forceps and removed using a scalpel

Care of products made from Latex Foam Rubber:

1. Latex foam is a natural product; store away from strong light and in an airtight bag to reduce exposure to ozone. Ozone is generated by some electrical equipment including telephones and computers.
2. Wash your hands before touching the foam. (If the foam is handled after contact with certain metals such as copper, it becomes yellow and discoloured).
3. The foam may be washed effectively using any soap and rinsed afterwards. 'Vanish'™ soap is very effective.
4. Do not ingest the product and practice normal hygiene after handling the products.

