

Ectopic Pregnancy

Part No: **60315** ⚡

For practicing the removal of an ectopic pregnancy using diathermy, laser and other monopolar electrocautery techniques and standard instruments.

Skills

- Surgical opening of the fallopian tube (salpingostomy)
- Removal of an ectopic pregnancy
- Surgical excision of a fallopian tube (salpingectomy)

Features


- Cost effective – contains three ectopic pregnancies
- Manufactured from Limbs & Things proprietary conductive material
- Provides a realistic response with instruments
- Recommended as the 'Gold Standard' for training following research conducted by two UK hospitals
- Use with MATT Trainer **50150**

Package supplied

- 1 'sheet' with 3 ectopic pregnancies

Other information

- Must be stored in a refrigerator until use to ensure maximum conductivity
- Must be used with a patient return electrode plate

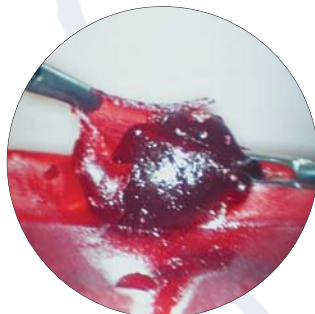

Endoscopic view of a Limbs & Things Ectopic Pregnancy being removed

Safety

- Follow instrument safety guidelines at all times.
- Use a standard or paediatric Disposable Patient Return Electrode to earth the product.
- Performing electro-surgery (diathermy) on any carbonaceous material will produce quantities of smoke. As when resecting human tissue, users must be provided with adequate protection to prevent the inhalation of fumes, such as extraction, personal respirators or similar.

Product Preparation

- In order to maintain maximum conductivity, remove the product from the refrigerator only a few minutes before use.
- Should product performance appear to be impaired, the simulator can be rehydrated in water for approximately 1-2 minutes immediately before use.

Using the Ectopic Pregnancy in the MATT Trainer

- 1** Suspend the Ectopic Pregnancy 'sheet' within the MATT Trainer by using the securing clips attached to the frame.
- 2** Place the Patient Return Electrode under the 'sheet' with the adhesive side up. Ensure that the bottom of the 'sheet' is in contact with the electrode. Ideally a strip at least 1cm wide, running along the entire width of the 'sheet', should be in contact with the electrode.
- 3** Attach the lead of the Patient Return Electrode to the electro-surgical unit.
- 4** Place the Abdominal Skin over the frame of the MATT Trainer and secure it in position with the four black turn clips at the corners of the base.
- 5** Ensure that you have adequate extraction before commencing electro-surgery.



Designed and manufactured by Limbs & Things Limited,
 Sussex Street, St. Philips, Bristol, BS2 0RA, UK
 Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
 E-mail: sales@limbsandthings.com www.limbsandthings.com