



# Laparoscopic Arm

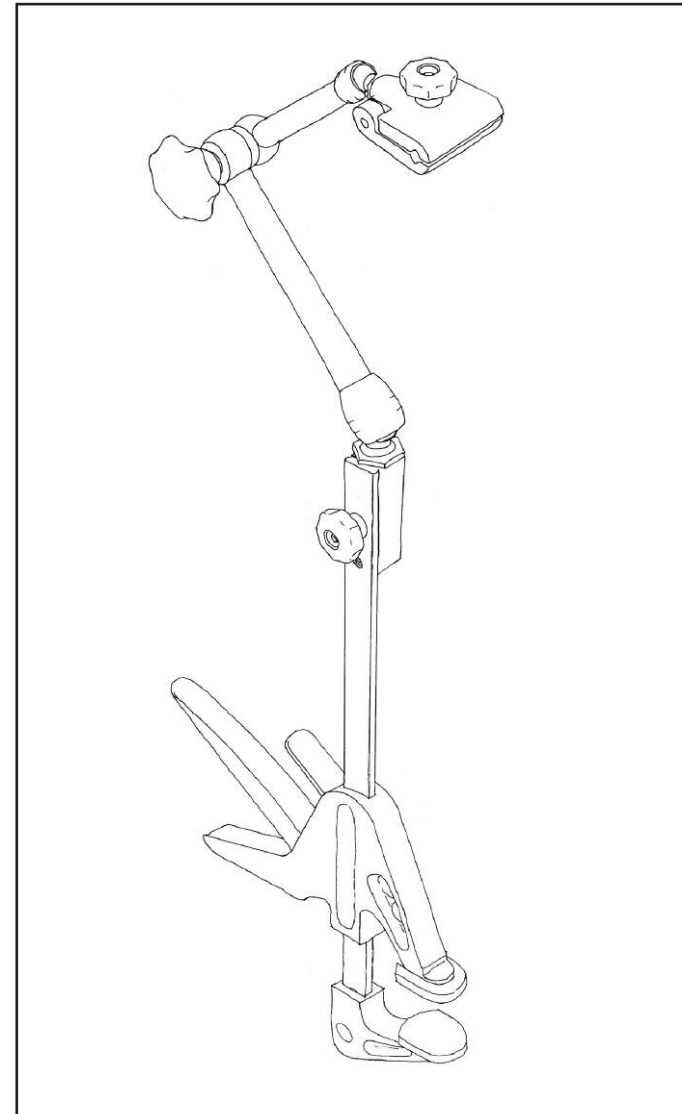

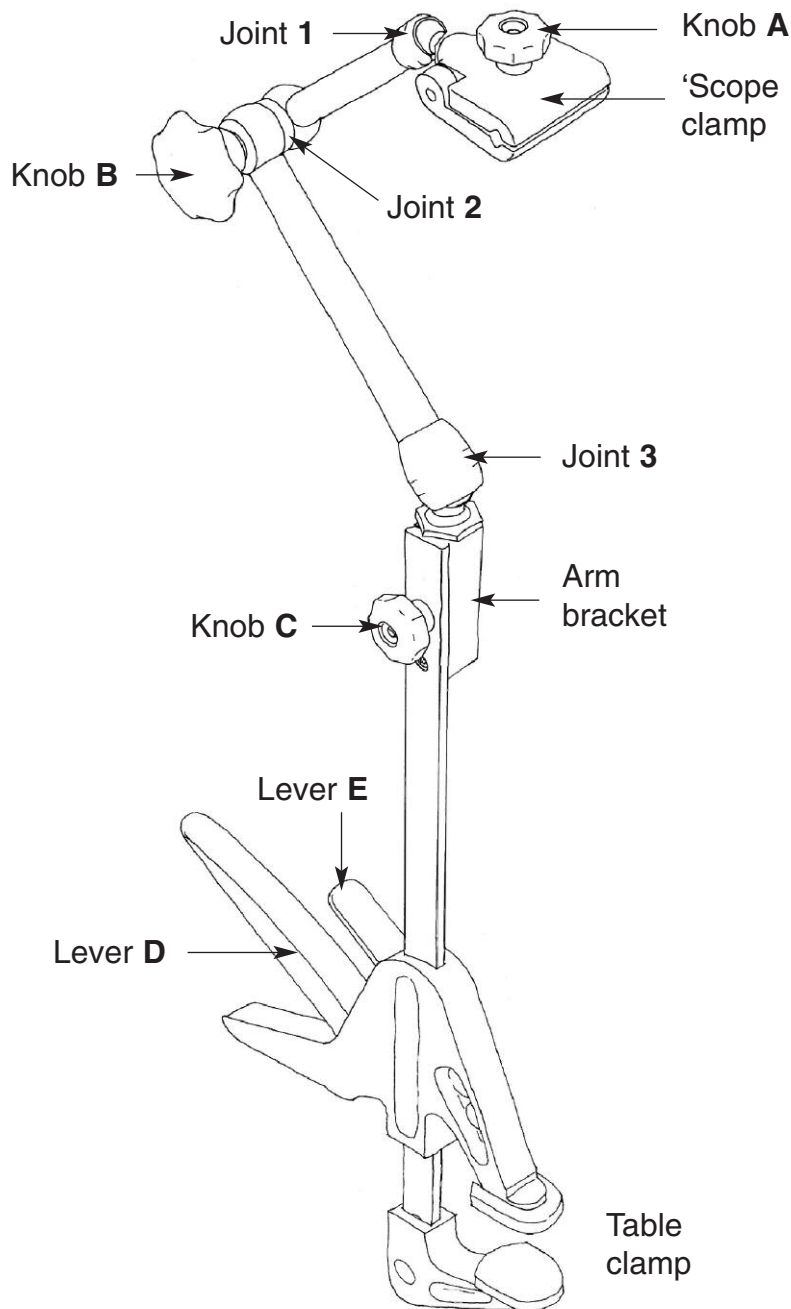

## Important:

The **Laparoscopic Arm 50147** 'scope clamp is designed to accommodate scope diameters from 5mm to 14mm.

This product is to be used for training purposes only.

## Operating instructions:

1. Knob **C** secures the arm bracket to the table clamp.  
For use: Position the arm bracket in the upright position as shown and secure by tightening Knob **C**.
2. Lever **D** tightens table clamp.
3. Lever **E** slackens table clamp.
4. Knob **B** simultaneously tightens joints **1**, **2** and **3**.  
Slacken Knob **B** slightly to allow the arm to be articulated.  
Position arm as required and re-tighten Knob **B**
5. Knob **A** operates the 'scope clamp.  
Slacken Knob **A** until 'scope can be accommodated between the grooves of the clamp.  
Insert 'scope and re-tighten Knob **A** to hold scope in place.

**NB: Take care not to over-tighten Knob A**

**NB: Always remove 'scope from clamp before slackening table clamp.**

6. For Storage: Slacken Knob **C** sufficiently to allow the arm bracket to turn through 180°. It may now be fitted into the case.