

Lumps & Bumps Set

Part No: **00125**

A variety of simulated pathologies to be used in conjunction with the 'Strap-on' Breasts **40100** when performing breast examination.

Skills

- Diagnosis of pathologies:
 - cyst
 - lipoma
 - fibroadenoma
 - carcinoma

Features


- Multiple simulated pathologies
- Material properties closely mimic real pathologies
- Can be placed in various positions between the soft tissue chest and the breast infills of the 'Strap-on' Breasts
- Reusable

Package supplied

- 3 lipomas
- 3 small cysts
- 3 carcinomas
- 2 small lumps
- 1 fibroadenoma

Positioning pathologies

1

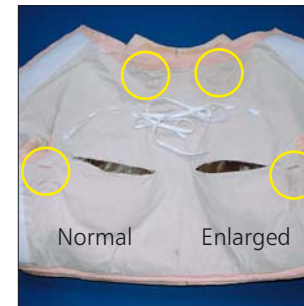

The Strap-On Breasts are supplied with lymph nodes above the clavicle and in the axillae: normal size on the left and enlarged on the right.

Pathologies supplied (from left to right):

- lipoma (yellow)
- cyst (white)
- carcinoma (translucent)
- small lumps (ball bearings)
- fibroadenoma (capsule with lump)

2



The Lumps & Bumps Set pathologies are positioned between the breast infill and the outer layer of the soft tissue chest. This space is accessed via the slits on the back of the soft tissue chest.

Choose the position of the pathology within the breast cavity and pull the breast infill back to allow access to the site.


3



Ensure the correct orientation of the pathology and put it in place.

Allow the breast infill to drop down onto the pathology.

4

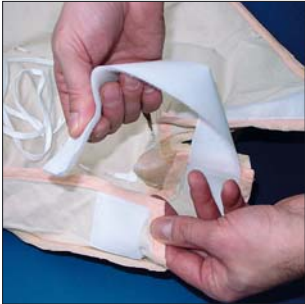

Press the breast infill down onto the pathology to keep it in position. Repeat the placement process as many times as is necessary.

To attach the soft tissue chest first ensure the shoulder straps are undone.


5



Next, place your hands underneath the breasts and support them to ensure that the pathologies are held in place.

Attach the soft tissue chest to the support vest as described in the 'Strap-On' Breast user guide. A second person will be required to attach the soft tissue chest whilst you hold it against you.

Lumps & Bumps Set User Guide

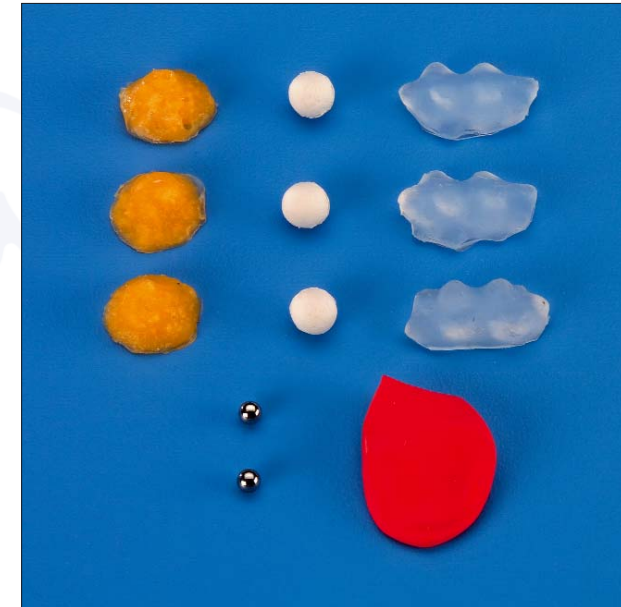

Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK.
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
sales@limbsandthings.com www.limbsandthings.com

Issue 1, March 2005
© 2005 Limbs & Things

For more skills training products visit

www.limbsandthings.com