

READ THIS BEFORE USE: Quick setup guide (See the User Guide for detailed instructions)


1 Removal



To remove the product from the case: with one hand grasp the torso end whilst supporting the buttocks with the other hand. Lift and remove.


The Baby is in the case is underneath.

2 Attach base

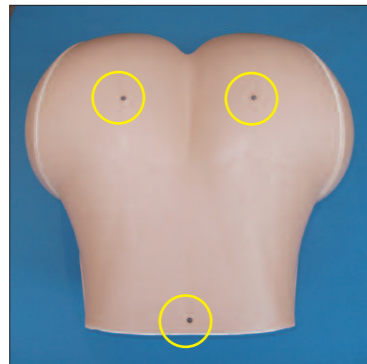

Support the Mother on the Baby case. Place the Base upside down on the back of the Mother.

Align the holes in the base with those on the back of the Mother.

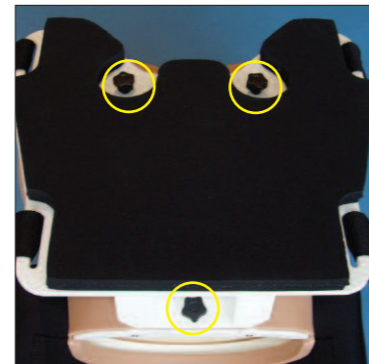


Fix the Base in position by tightening the single turn knob.

Secure the Base in place by tightening the remaining two. Ensure all three turn knobs are secure.


DO NOT overtighten the knobs

3 Secure straps

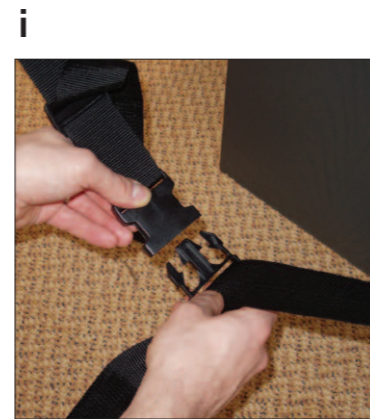

Draw the ends of one of the pair of straps together underneath the work surface and clip together.

Tighten by pulling the loose end.

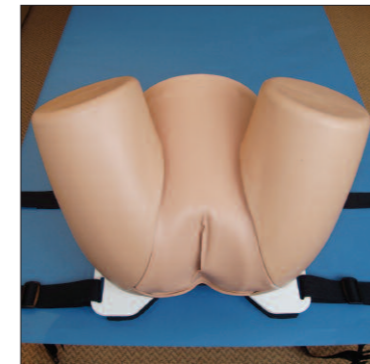

Ensure both sets of straps are fully tightened.

The Mother should now be firmly secured to the work surface or delivery bed and ready to use.

4 Lubrication



Using the lubricant provided, spray the perineal muscles 2 or 3 times.



Spread the lubricant evenly over the surfaces of the muscles, both front and back.

Spray the Baby 2 or 3 times.

Spread the lubricant over the head and upper body.

DO NOT use the model without lubrication as damage may occur


5 Position Baby

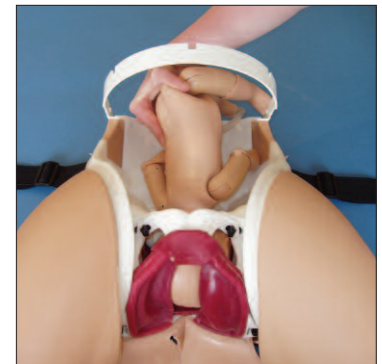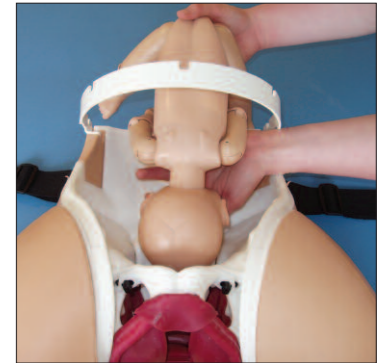

With the Baby's arms and legs fully extended, position the Baby face down (OA) in the pelvis.

Advance the head until it is engaged in the pelvis. Ensure the shoulders are in the oblique position.

For more detailed set up instructions of all birthing positions see the User Guide.

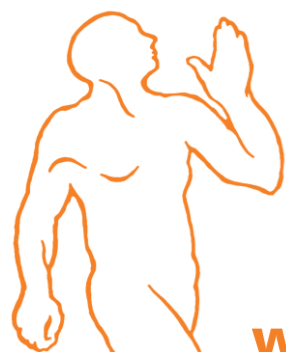

PLEASE TURN OVER

Quick setup instructions on the other side