

Skin Pad Jig Strap: Attaching to the torso

1



Position the Strap behind the torso with the stiff Velcro® tab in the right hand.

Draw the other long end of the Strap across the front of the torso.

2



Keep the long end of the Strap under tension and pull the stiff Velcro® tab onto the torso securing it in place.

Ensure that the area where the Skin Pad Jig will be attached is soft Velcro® only. Line up the Velcro® pads on the underside of the Skin Pad Jig with the Strap.

3



Press the Jig down firmly onto the Strap. The Jig is now ready to use. Skin Pads can be changed whilst the Jig is attached to the Strap.

For extra realism the Skin Pad can be correctly draped so that it appears to be the real skin of the trainee.

Skin Pad Jig Strap User Guide



Designed and manufactured by Limbs & Things Limited,
Sussex Street, St. Philips, Bristol, BS2 0RA, UK.
Telephone: +44 (0)117 311 0500 Fax: +44 (0)117 311 0501
sales@limbsandthings.com www.limbsandthings.com

Issue 1, March 2005
© 2005 Limbs & Things

For more skills training products visit

www.limbsandthings.com

Skin Pad Jig Strap

Part No: **00510**

This low cost addition for the Skin Pad Jig Mk 2 **00500** provides a simple solution for wearing Limbs & Things skin pads when combining technical skills with professional-to-patient communication.

Also fits Skin Pad Jig **00131** (which is no longer available. Pre 2005) when used with Skin Pad Jig Fixing Kit **00520**.

Skills

- Professional-to-patient communication

Features


- Double-sided Velcro™ strap long enough for attaching to the limbs or body of a trainee
- Supplied in convenient packs of 3 for group skills training

Package supplied

- 3 Skin Pad Jig Straps

Skin Pad Jig Strap: Attaching to a limb

1

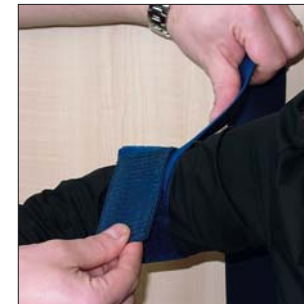

Loop the Skin Pad Jig Strap under the arm. (This procedure is shown being carried out on the right arm of the trainee).

Pull the long end of the Strap back over the top of the arm. Pull the stiff Velcro® tab forwards over the arm, securing it onto the strap.

2



Rotate the strap so that the stiff Velcro® tab is underneath the arm. Ensure that the area where the Skin Pad Jig will be attached is soft Velcro® only.

Line up the Velcro®^{j1} pads on the underside of the Skin Pad Jig with the Strap.

3



Press the Jig down firmly onto the Strap. The Jig is now ready to use. Skin Pads can be changed whilst the Jig is attached to the Strap.

The Strap and the Skin Pad Jig can also be attached to the leg.

For extra realism the Skin Pad can be correctly draped so that it appears to be the real skin of the trainee.